

St. Jude Medical is focused on reducing risk by continuously finding ways to put more control into the hands of those who save and enhance lives.

1. Data on file, St. Jude Medical.
2. As referenced in the Merlin PCS Reference Manual.

ATRIAL FIBRILLATION    CARDIAC RHYTHM MANAGEMENT    CARDIAC SURGERY    CARDIOLOGY    NEUROMODULATION

**Global Headquarters**  
One Lillehei Plaza  
St. Paul, Minnesota 55117  
USA  
+1 651 483 2000  
+1 651 490 4310 Fax

**Cardiac Rhythm Management Division**  
15900 Valley View Court  
Sylmar, California 91342  
USA  
+1 818 362 6822  
+1 818 364 5814 Fax

**St. Jude Medical Sweden AB**  
Veddestavägen 19  
SE 17584 Järfälla  
Sweden  
+46 8 474 4147  
+46 8 760 5126 Fax

**St. Jude Medical Europe, Inc.**  
The Corporate Village  
Figueras Building  
Avenue Da Vinci laan 11 Box F1  
B-1935 Zaventem  
Belgium  
+32 2 774 68 11  
+32 2 772 83 84 Fax

**St. Jude Medical Brasil Ltda.**  
Rua Frei Caneca, 1380  
7º ao 9º andares  
01307-002 - São Paulo (SP)  
Brazil  
+55 11 5080 5400  
+55 11 5080 5423 Fax

**St. Jude Medical (Hong Kong) Ltd.**  
Unit 2701-07 27/F, COSCO Tower  
Grand Millennium Plaza  
183 Queen's Road  
Central, Hong Kong  
+852 2996 7688  
+852 2956 0622 Fax

**St. Jude Medical Japan Co., Ltd.**  
3-1-30, Minami-Aoyama  
Minato-ku  
Tokyo 107 0062  
Japan  
+81 3 3423 6450  
+81 3 3402 5586 Fax

[sjm.com](http://sjm.com)



**Brief Summary:** Prior to using these devices, please review the instructions for use for a complete listing of indications, contraindications, warnings, precautions, potential adverse events, and directions for use.

Devices depicted may not be available in all countries. Check with your St. Jude Medical representative for product availability in your country.

Unless otherwise noted, <sup>TM</sup> indicates that the name is a trademark of, or licensed to, St. Jude Medical or one of its subsidiaries. ST. JUDE MEDICAL, the nine-squares symbol and MORE CONTROL. LESS RISK. are trademarks and service marks of St. Jude Medical, Inc. and its related companies. ©2009 St. Jude Medical. All Rights Reserved.


Item no. XXXXXXXXX

Anthem<sup>TM</sup>  
CRT-P



## INTELLIGENT PACING FOR SIMPLIFIED PATIENT MANAGEMENT.

The Anthem™ device from St. Jude Medical significantly simplifies patient management—from implant through follow-up. With industry-leading longevity, interrogation speeds that streamline in-clinic follow-ups, and the intelligence to sense and automatically adjust device parameters, the Anthem CRT-P is designed to maintain effective therapy and enhance patient safety.

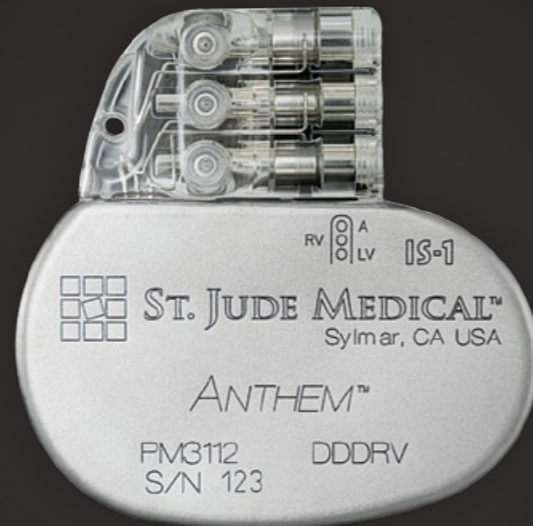

### Streamlined Follow-ups

Faster access to key diagnostic data.

- Daily automatic measurements may eliminate the need for in-clinic testing.
- Faster telemetry means interrogations are up to three times faster to help reduce follow-up times.<sup>1</sup>

### Daily Automatic Measurements

	Atrium	RV	LV
Capture	✓	✓	✓
Sensing	✓	✓	✓
Impedance	✓	✓	✓

### Intelligent Optimization

Simple solutions deliver optimal therapy based on patient needs.

- Comprehensive atrial arrhythmia management tools help ensure consistent therapy delivery.
  - AT/AF alerts can be set to notify patients and their clinics when a programmed threshold has been exceeded.
  - AF Suppression™ algorithm is clinically proven to suppress episodes of paroxysmal and persistent AF.<sup>1</sup>
  - Triggered Pacing with BiV Trigger Mode promotes continuous BiV pacing, especially during AF, by triggering pacing in both ventricles in response to a sensed ventricular event.
- VectSelect™ programmable LV pulse configuration has four LV pacing vectors and allows lead adjustment to take place non-invasively using the programmer, saving time during and after implant.
- Negative AV Hysteresis with search helps minimize the risk of losing BiV pacing by automatically reducing AV delay when intrinsic activity is present.
- QuickOpt™ Timing Cycle Optimization provides quick and effective AV and V-to-V optimization at the touch of a button.

### Enhanced Safety and Reliability

Designed to provide confidence based on a commitment to patient safety.

- Programmable audible alerts can notify patients to changes in device performance.
- Industry-leading longevity of 7.9 years<sup>2</sup>—with stored electrograms turned on—backed by a five-year warranty.
- Multiple hardware and software system safeguards ensure therapy delivery, even in the unlikely event of individual component failures.

St. Jude Medical is committed to putting more control into the hands of physicians and thereby reducing risk. As part of a comprehensive portfolio that delivers on this promise, the Anthem CRT-P, backed by 50 years of pacing experience, provides intelligent sensing and adjustment of selected device parameters to help streamline follow-ups, simplify therapy optimization and enhance clinic efficiency.